

Diversey™

Oxivir Plus

Cleaner & disinfectant for non-porous hard surfaces and non-invasive medical devices

Description

Diversey Oxivir Plus is a broad spectrum cleaner and disinfectant for non-porous hard surfaces and non-invasive medical devices.

Key properties

- Bactericidal, Virucidal, Yeasticidal, Fungicidal and Sporicidal
- Patented Hydrogen Peroxide based acidic formulation
- Short contact times
- Effective Cleaner
- Fragrance free

Benefits

- Extensively tested against European norms and effective against multiple pathogens of concern, including HIV, HBV, HCV, Avian Influenza, Pseudomonas aeruginosa, Staphylococcus aureus, Salmonella enterica, MRSA, VRE and Norovirus
- Cleans and disinfects in one step, reducing working time
- Excellent cleaning performance, including descaling and lime limitation removes the need for a pre-cleaning step
- Good user safety profile, with no hazard warnings on RTU format; does not contain respiratory irritants
- AHP formulation is gentle on surfaces extending equipment life
- Good environmental profile. Active breaks down into oxygen and water

Use instructions

Spray cleaning and disinfection:


1. Remove gross soils.
2. Apply to surfaces by either spraying the wiping cloth and wiping surfaces or applying directly to the surface through use of a trigger sprayer.
3. After the contact time, either wipe dry or allow to air dry.

Bucket cleaning and disinfection:

1. Remove gross soils.
2. Apply to surfaces with mop/cloth.
3. After the contact time, either wipe dry or allow to air dry.
4. Change the solution if it becomes visibly soiled or at least every 24 hours

Dosage:

3.5% to achieve disinfection claims made (35mL for 1L solution).

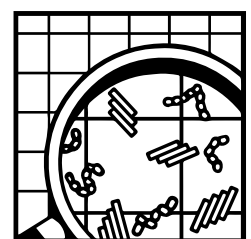

© A.I.S.E.

**This dosage is according to optimal conditions, recommendations may vary, please consult with your Diversey representative for directions.*



Oxivir Plus

Technical data

Appearance: Clear, yellow liquid

pH value (neat): 0.5

pH value (in use): 1.8 +/- 0.5 at 3.5% dilution

Relative density (20°C): 1.029

The above data is typical of normal production and should not be taken as a specification.

Safe handling and storage information

Full guidance on the handling and disposal of this product is provided in a separate Safety Data Sheet; sds.diversey.com. Store in original closed containers away from extremes of temperature. Only for professional users / specialists.

Reacts with oxidisable substances.

Product compatibility

Do not mix with other products. Do not use warm water. Do not use on water- or acid-sensitive surfaces. Under recommended conditions of use, Diversey Oxivir Plus is suitable for use on most materials commonly encountered.

Microbiological data

Efficacy proven by compliance with European norms. VAH listed.

Environmental information

The surfactants used in this product are biodegradable in compliance with the requirement of EU Detergent Regulation, EC 648/2004.

Available pack sizes

Oxivir Plus is available in 6x1L, 2x5L, SmartDose 1x1.4L and J-Flex 1x1.5L.